

## Harmony Platform

### SR Module with Advanced Fluorescence Technology

The **SR Module of the Harmony Platform** provides innovative technology for non-ablative skin rejuvenation. It employs proprietary AFT pulsed light technology in the yellow wavelength (570-950nm) along with adjustable parameters for effective treatment of vascular and pigmented lesions.

- **Fast** - large spot size (6.4 cm<sup>2</sup>) enables rapid treatment
- **Easy to use** – ergonomic handpiece
- **Treat flushing, erythema, rosacea, Poikiloderma of Civatte, lentigos, ephelides, pores, telangiectasias**
- **No downtime** – safe and efficacious results

#### The Most Advanced Light-Based Technology for Photorejuvenation

To achieve precise, selective absorption of the light energy the SR handpiece employs two novel electro-optical components:

#### Advanced Fluorescence Technology (AFT™)

A special filtering system converts the unused shortwave light into the visible yellow spectrum (570-950nm), increasing spectral emission and penetration, without excess energy levels for safer, more effective treatments.

#### Equally Distributed Fluence (EDF™)

An innovation in electronics which enables continuous square-shaped pulsing for moderate peak power throughout the entire pulse, for efficacious, safe results.




# Harmony




## Harmony System: Multi-Application, Multi-Technology Platform

- The most versatile, expandable light and laser system on the market.
- 10 different treatment heads for numerous combination therapy protocols
- 8 distinct technologies:
  - ▶ **AFT**
  - ▶ UVB
  - ▶ 532nm Fd Q-Switched Nd:YAG
  - ▶ 1320nm Nd:YAG
  - ▶ Near-infrared light
  - ▶ 1064nm LP Nd:YAG
  - ▶ 1064nm Q-Switched Nd:YAG
  - ▶ 2940nm Er:YAG
- Over 60 FDA cleared indications, including:
  - ▶ **Skin rejuvenation**
  - ▶ Acne
  - ▶ Vascular lesions
  - ▶ Hypopigmentation
  - ▶ Skin resurfacing
  - ▶ Tattoo removal
  - ▶ Hair removal
  - ▶ Acne scarring
  - ▶ Pigmented lesions
  - ▶ Laser peeling
  - ▶ Cutaneous lesions
  - ▶ Fine lines and wrinkles
- Customizable and upgradeable
- Plug&Play technology – no downtime when upgrading
- Low cost of ownership – high return on investment
- Easy to use – selected preprogrammed parameters

Before

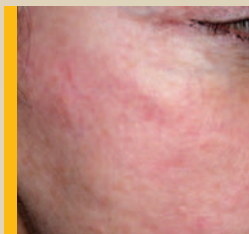

After

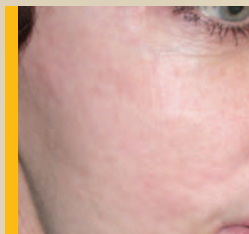


Photos Courtesy: BCK Patel M.D. FRCS, Professor Chief, Division of Facial, Orbital and Oculoplastic Surgery, University of Utah, Salt Lake City, Utah, USA  
AFT 540 VP, 570 SR Modules after 5 Tx. 14-17 J/cm<sup>2</sup> @ 10 msec.

Before



After

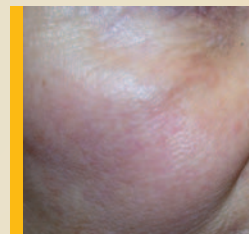

Photos Courtesy: BCK Patel M.D. FRCS, Professor Chief, Division of Facial, Orbital and Oculoplastic Surgery, University of Utah, Salt Lake City, Utah, USA  
AFT 540 VP, 570 SR Modules after 5 Tx. 15-18 J/cm<sup>2</sup> @ 10 msec.

## Technical Specifications of SR Module

|                                      |                                                   |
|--------------------------------------|---------------------------------------------------|
| Light Source                         | Pulsed Light with AFT and EDF                     |
| Spectrum                             | 570-950nm                                         |
| Energy Density (Fluence)             | 5-20 J/cm <sup>2</sup>                            |
| Treatment Area (Spot Size)           | 40x16 mm (6.4 cm <sup>2</sup> )                   |
| Pulse Duration (Sec.)                | 10, 12, 15                                        |
| Pulse Repetition Rate                | 1/3 Hz                                            |
| <b>Harmony System Specifications</b> |                                                   |
| Electrical                           | 100-120 VAC, 5A, 50/60 Hz                         |
|                                      | 220-240 VAC, 5A, 50/60 Hz                         |
| Weight                               | System: 88 lbs.(40 kg.)<br>Cart: 77 lbs. (35 kg.) |
| System Dimensions                    | 26" x 18" x 16" (65 x 45 x 40 cm)                 |
| Cart* Dimensions                     | 22" x 22" x 30" (55 x 55 x 75 cm)                 |

\* Optional

[www.almalasers.com](http://www.almalasers.com)



CE 0473



©2006, Alma Lasers, Ltd. All rights reserved. Alma® Lasers Ltd., its logo, Accent®, Aria™, Harmony®, Sonata®, Soprano®, DualChill™, AFT™, EDF™, Magic of Light® and BlendMode™ are the trademarks of Alma Lasers, Ltd. Product specifications are subject to change without notice.

**United States**  
6555 NW 9<sup>th</sup> Avenue  
Suite 303  
Fort Lauderdale, FL 33309  
Tel: (954) 229-2240  
Fax: (954) 229-8310

**Headquarters**  
7 Haeshel St., P.O.B 3021  
Caesarea Industrial Park  
Caesarea, Israel 38900  
Tel: +972-4-627-5357  
Fax: +972-4-627-5368

**Alma Lasers™**  
Wellbeing Through Technology